

AOEL®

Age of Elegance



ATELOCOLLAGEN **3%**

COLLAGEN-i ATELOCOLLAGEN 3%




PACKAGING 1 ml × 1 syringe

INGREDIENT AteloCollagen 30mg/ml (30mg/syringe)


SKIN CONCERNS



- Mitigate dark circles under the eyes
- Diminish fine lines around the eyes
- Correct tear trough imperfections
- Address sagging and sunken cheeks
- Replenish collagen
- Reduce fine lines
- Promote collagen regeneration
- Minimize pores visibility

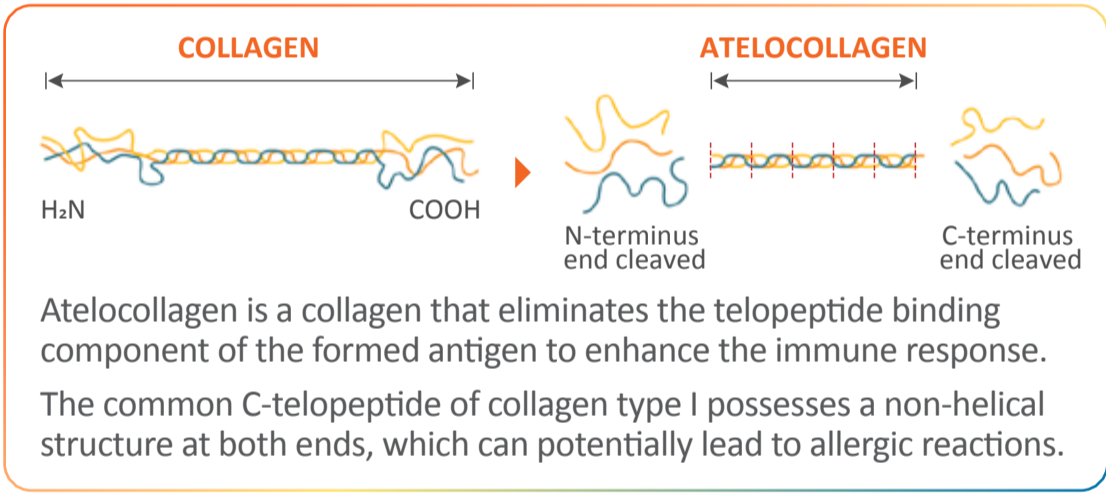

- ✓ Comprising a single ingredient of 3% pure collagen for enhanced safety, replenishing, and promoting natural collagen production
- ✓ Collagen-i ensures a natural appearance with results lasting up to 2 years after a treatment course, followed by natural absorption
- ✓ Enhancing cellular functions in both the epidermis and dermis
- ✓ Improving skin texture and achieving optimized results upon completing a treatment course

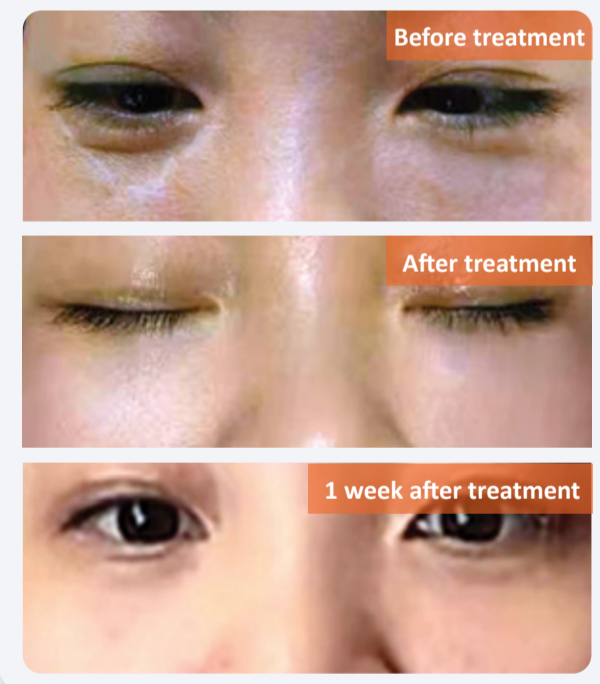

ATELOCOLLAGEN

*Restore Collagen
Resurge Elasticity
Reinforce Firmness*

COMPARISON

DIFFERENCE BETWEEN COLLAGEN AND HYALURONIC ACID

COLLAGEN



Collagen can enhance wrinkle reduction by improving the treatment area and promoting the formation of new soft tissue, thus extending the duration of the collagen filling effect



Enhances tissue regeneration and refines skin texture



Minimizes the risk of migration



long-lasting results

HYALURONIC ACID



yaluronic acid is highly effective in achieving filling effects because it can bind substantial amounts of water molecules. However, its results may not last as long, and it may not yield a natural appearance



Excellent water retention capability



Effective for filling and contouring



shorter-lasting results

DIFFERENCE BETWEEN PURE COLLAGEN, COLLAGEN BOOSTER AND AUTOLOGOUS FAT GRAFTING

COLLAGEN



Atelocollagen is a highly purified type I collagen derived from porcine sources. It can be completely decomposed and absorbed by the body. However, the extraction process for pure collagen is challenging, and it is not resistant to high temperatures. Therefore, it is essential to consult with a doctor to adjust the treatment sequence if there is a requirement to use it in procedures like radiofrequency treatment or any other high-temperature treatments



Composed of pure collagen for safety



Provides a natural look without migration



Enhances skin texture



A complete treatment course is necessary

COLLAGEN BOOSTER



Collagen boosters, also known as collagen stimulators or biostimulators, stimulate natural collagen production. Ideal for larger areas, they require skilled dermatologists for effective administration, with care to avoid superficial injection. Note that around 70% to 80% absorption is typical



Provides an ideal filling effect



Requires a higher dosage



Remains insoluble



Long-lasting effects

AUTOLOGOUS FAT GRAFTING



Autologous fat grafting is suitable for filling larger areas and demands a higher level of expertise from physicians. The survival rate of grafted fat tends to be low, and the distribution of fat in the body is predetermined by one's gender and genetics. Generally, fat tends to accumulate in gender-specific patterns and specific areas. The duration of maintenance also varies based on the physician's experience and the treatment plan



Requires a higher level of technical expertise



Associated with a higher risk of adverse effects



Suitable for large depressed areas



Has a low survival rate for grafted fat

COLLAGEN

LIFESTYLE HABITS THAT CONTRIBUTE TO COLLAGEN LOSS



High sugar diets



Lack of sleep




Intake of too much greasy and processed food




Unprotected sun exposure

The key to help reduce collagen loss is


IMPROVE LIFESTYLE HABITS

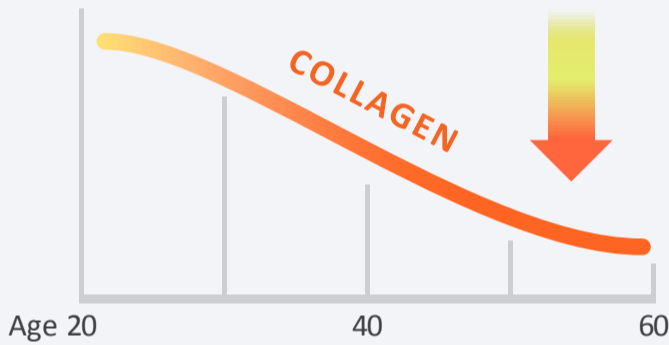


During childhood, collagen production reaches its peak, playing a crucial role in maintaining the skin's strength and elasticity, resulting in a supple and youthful appearance. However, as we age, our skin's natural collagen production starts declining, decreasing by approximately 0.5% per year. This reduction in collagen levels can lead to a loss of skin elasticity, which impacts its overall appearance and contributes to an older look.

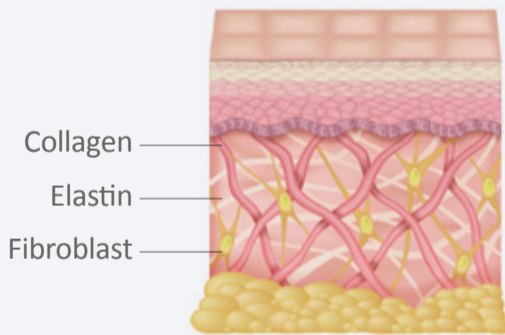

High density of collagen and elastin

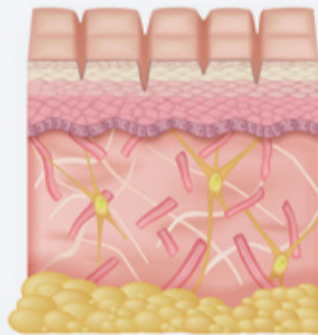

Degrading collagen and elastin

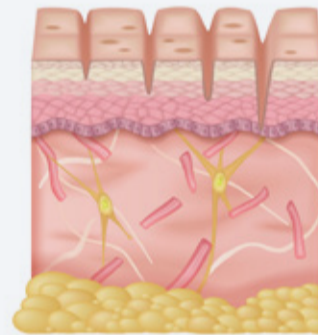


Lack of collagen and elastin

Epidermis
Dermis
Hypodermis

INDIVIDUALS SUITABLE FOR COLLAGEN TREATMENT



Individuals concerned about post-surgery adverse effects

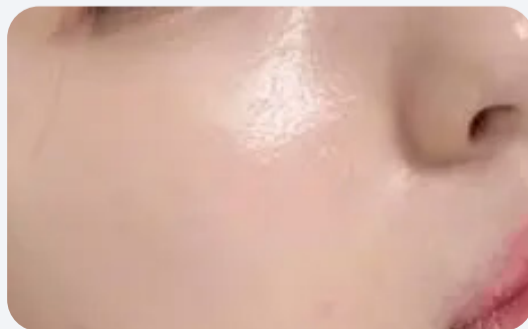


Those seeking the highest level of natural beauty



People desiring a natural filling effect and improved skin texture

INSTRUCTION

USAGE GUIDELINES

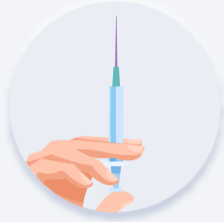

Injection




Radio Frequency (RF) Microneedling
(RF LASER / SECRET)




Radiofrequency Microneedling
(POTENZA)

APPLICATION AREA

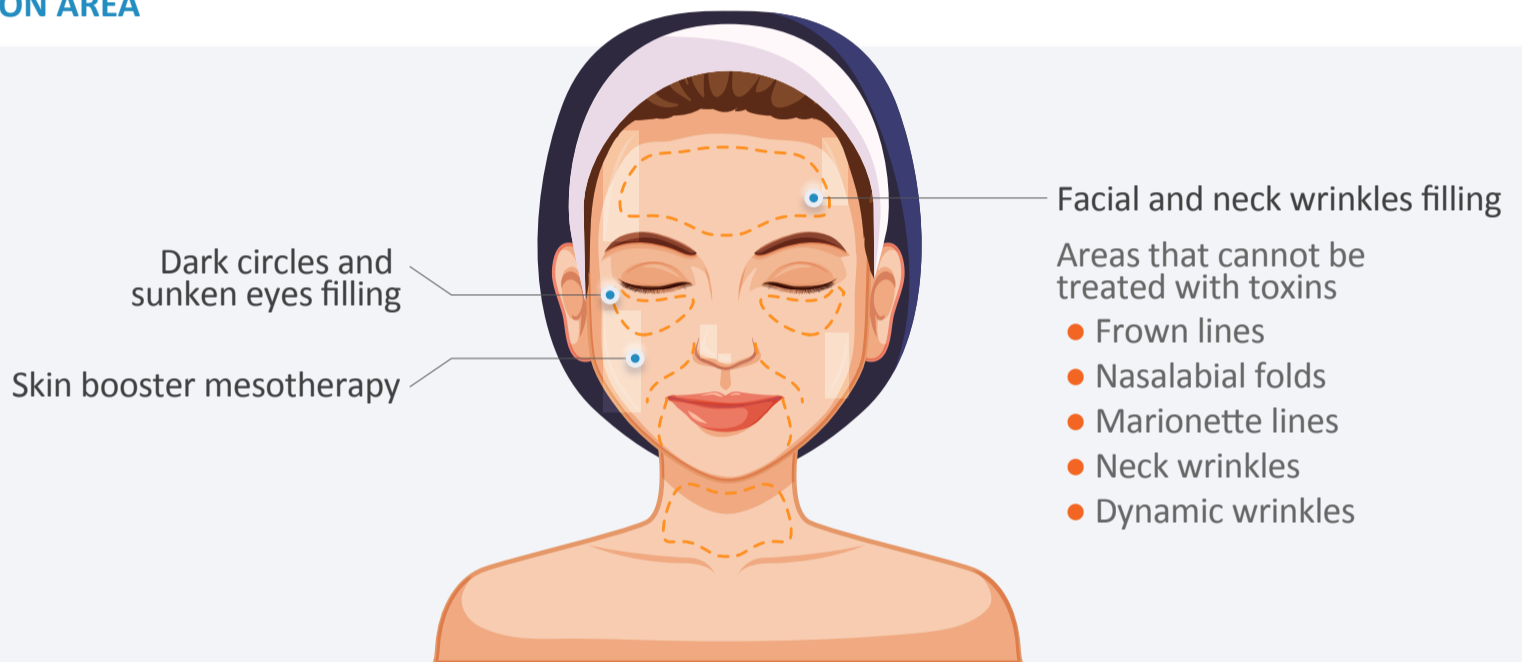

TREATMENT PROGRAM

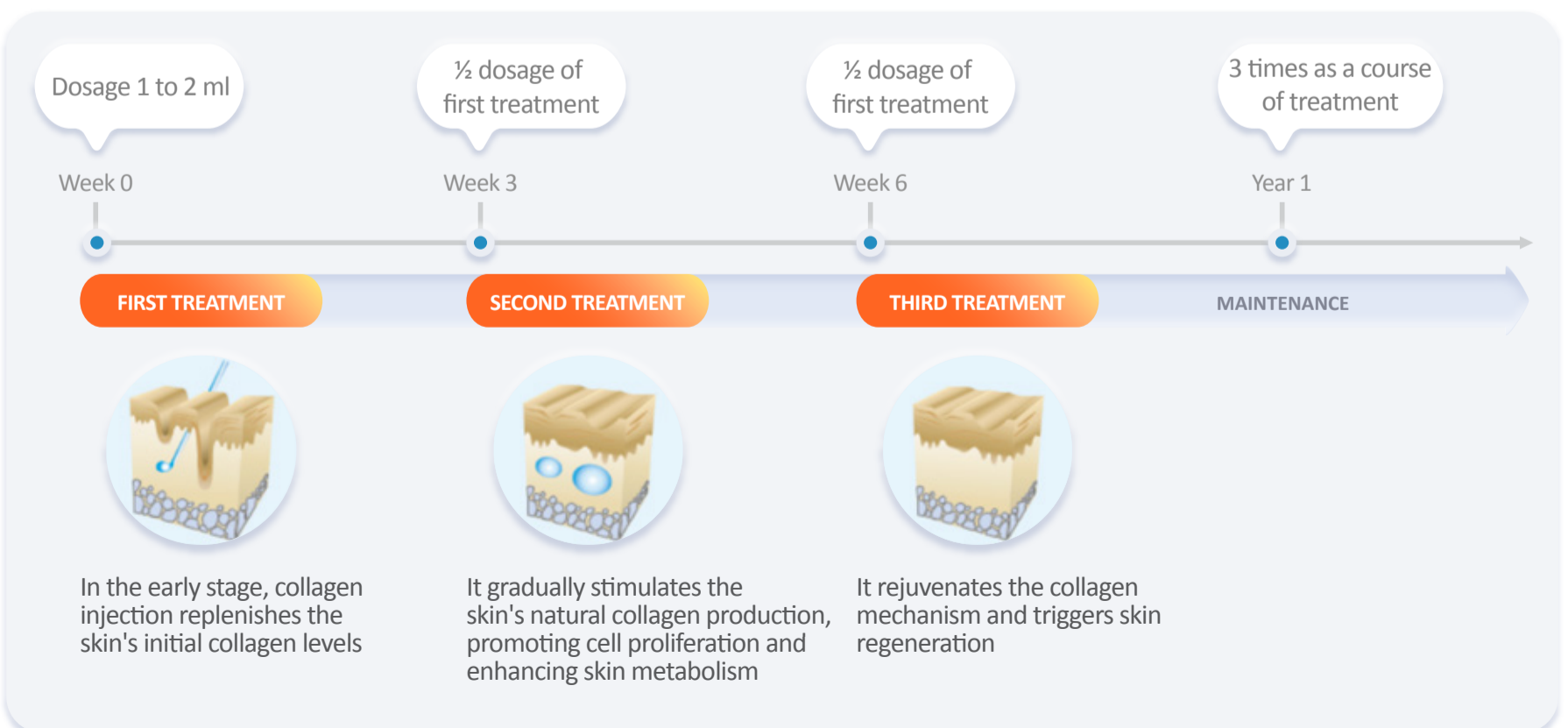

INSTRUCTION

INSTRUCTION OF USE

WRINKLES

- Forehead wrinkles
- Frown lines
- Crow's feet
- Nasolabial folds
- Marionette lines

SUNKEN FOREHEAD
COLLAGEN-*i* + HA

AROUND THE EYES
-Dark circles
-Lifting
COLLAGEN-*i* 1ml + Saline 1 to 2ml

SUNKEN CHEEKS
COLLAGEN-*i* + HA

NECK LINES

SKIN BOOSTER PROGRAM :

Basic
●○○

COLLAGEN-*i* 1ml + Saline 3 to 4ml

OR

COLLAGEN-*i* 1ml + Toxin 20U + Saline 3ml

Sole
●●○

COLLAGEN-*i* 3ml

Injection point : 1 cm interval
Injection depth : SUB-Q
Application cycle : 1 to 1.5 month(s) is the best recommendation
Maintenance period : The effect can last for up to 1 year

Advance
●●●

Toxin 12U + COLLAGEN-*i* 0.5ml + Saline 1ml

OR

Toxin 12U + COLLAGEN-*i* 1ml + Saline 0.8ml

OPT 1 Nourishing product 0.5ml + OPT 2 Repairing product 0.5ml

For Mild Case

COLLAGEN-*i* 0.5ml + Lidocaine 0.5ml + Saline 1ml

For Severe Case

2 types of layers (according to wrinkle types)

OPT 1 Deep Layer
COLLAGEN-*i* 0.5ml + Saline 1ml

Superficial Layer
COLLAGEN-*i* 0.5ml + Saline 1.5ml

OPT 2
*If there is any remaining amount after treating neck lines, it can be utilized for skin booster treatment.
COLLAGEN-*i* 1ml + Saline 3ml

F.A.Q.'S AFTER SURGERY

CAUSES AND TREATS OF REDNESS AND SWELLING AFTER SURGERY :



Cold packs, helps to constricts blood vessels to the area




Prescription antihistamines, if allergic occurs



Do not excessive dosing in one time



Be cautious of allergic to ingredients or surgical supplies

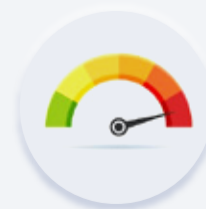

Post-Injection Inflammation. Avoid aggressive application

ADEL[®]

PRODUCT LINE-UP

RECOVERY Series

MASK

SKIN BOOSTER Series

MOISTURIZING HA 10
AMPOULE

SKIN BOOSTER
AMPOULE

SKIN REJUVENATING
AMPOULE

HEALING Series

HAIR
SOLUTION ES

SKIN
REJUVENATING ES
AMPOULE

FIRMING Series

FACE SOLUTION
ESSENCE

BODY SOLUTION
ESSENCE

COLLAGEN Series

COLLAGEN-*i*



Age of Elegance